



Disinfection- and cleansing plan for the dental lab

WHAT	HOW	USING	WHEN	WHO
Maintenance of object	Kinds of maintenance	Product (e.g. Disinfection solution)	Time, Stepchart, Sequence of hygenical procedure	Name and signature of responsible person

SPECIAL ENVIRONMENT/MISCELLANEOUS

Impressions (e.g. elastomere, alginate, hydrocolloids etc.)	Disinfection Rinse with water	Disinfection solution for impressions EuroSept Max Impression Liquid Concentration: undiluted Working time: 10 Min.	<ul style="list-style-type: none"> Directly after removing out of transport package 	All persons involved with the preparation of medical devices.
Contaminated dentures Bridges etc., gypsum models, bite registration models, shellac or plastic	Disinfecting and cleansing by immersing in solution or Ultrasonic cleaner.	Disinfetion solution for dentures Eurosept Max Impression Liquid Concentration: undiluted Working time: 10 Min.	<ul style="list-style-type: none"> Directly after removing out of transport package 	
Impression trays	If necessary post-clean.	Tray Cleaner Concentrate Concentration: 5 – 10 % Working Time: 45 Min.	<ul style="list-style-type: none"> Separate after using 	
Contaminated Laundry or clothing	Collect, disinfect, clean (e.g. by washing clothing hot, separate from private clothing).	e.g. using disinfecting detergent at a washing temperature of 95°C or 60°C	<ul style="list-style-type: none"> Whenever needed 	
Waste	Garbage must be disposed by avoiding pointy, sharp and breakable parts.		<ul style="list-style-type: none"> Whenever needed or daily 	

INSTRUMENTS

Instruments, e.g. handinstruments	Disinfection and cleansing by immersing in solution or in Ultrasonic cleaner.	Disinfection solutions for instruments EuroSept Max Instrument Lemon Concentration: 3 % 1,5 % Working time: 15 Min. 60 Min.	<ul style="list-style-type: none"> After conditioning of contaminated prosthodontics 	All persons involved with the preparation of medical devices.
Rotating instruments (bristle brushes, TC burs, endo files etc.)		EuroSept Max bur Concentration: undiluted Working time: 30 Min.	<ul style="list-style-type: none"> Whenever needed 	

SURFACES

Surfaces and floors, Work and storage surface, if necessary walls	Disinfection by wet-wiping. Refrain from using surfaces until completely dry.	Disinfection solution for surfaces EuroSept Max SurfaceConcentrate Concentration: 0,5 % Working time: 60 Min.	<ul style="list-style-type: none"> Whenever needed or daily Directly at viewable contamination 	All persons/cleaning staff
Transport trays	Disinfection and cleansing by spraying and wiping (full usage).	EuroSept Max Surface lemon Concentration: undiluted Working time: 2 Min.	<ul style="list-style-type: none"> At the latest before re-using 	
Accessories (e.g. articulators, face bow)	Disinfection by spray.	Disinfection by spray, exclusively with alcoholic active ingredients EuroSept Max Surface lemon Concentration: undiluted Working time: 2 Min.	<ul style="list-style-type: none"> Directly when received 	

HAND CARE

Compulsory is an easy-to-reach place where you can wash your hands, with running warm and cold water, including soap dispenser with cleansing solution, hand disinfection solutions and other related specific hand care products such as disposable hand towels. Fingers, hands and underarms should be free of jewellery such as rings, bracelets and watches. Dispensers and water tap should be hand-free operative.

Hands Washing (cleansing)	Hand cleansing: Below running water, rub with fluid cleansing solutions from a soap dispenser, dry-clean with a disposable hand towel.	Fluid cleansing solution: EuroSept Max Washlotion or EuroSept Max WashlotionSensitive Dry-clean with a disposable hand towel (box, dispenser).	<ul style="list-style-type: none"> Before work Whenever needed Before every meal After using the lavatory (box, dispenser) After work 	All persons
Disinfection	Hygenical Hand disinfection: Apply hand disinfection solution, rub thoroughly until solution dries up according to regulation CEN/EN 1500 (approx. 30 sec.). Hands must be air-dry before putting on gloves according to regulation CEN/EN 1500.	Hand disinfection from dispenser: Eurosept Max Hand Disinfection Dosage: 1x 3ml Working time: 30 sec.	<ul style="list-style-type: none"> Whenever needed and after contact with impressions and prosthodontics of patients 	
Hand care (cleansing)	Hand care and Hand cleansing: Avoid imperilling activity, until hand care (cleansing) is fully absorbed. Pay attention to rubbing areas between the fingers, cuticles and backhand areas.	Hand care solution: Hand cream Wear safety gloves; make an indossulability test. Skin reactions resulting from continuously wearing gloves require special solutions.	Hand cream: <ul style="list-style-type: none"> Before work After each break Whenever needed After work Suitable gloves: <ul style="list-style-type: none"> Wear gloves only while performing skin imperilling activites 	

BEHAVIOUR IN CASE OF DANGER AND CONSPICUOUS SKIN CHANGES:

Getting in contact with a product damaging the skin:

- Remove moistened clothes immediately
- Clean moistened bodily parts extensively with _____!

Contact person: Mr/Mrs _____

Tel.: _____

If there are conspicuous skin changes, call on a company doctor or a general practitioner!

Please complete the disinfection and cleansing plan for the dental lab with necessary information from the company.