INSTRUCTION FOR USE

SoftLine

Self Curing Soft Chairside Reline Material

DESCRIPTION

Henry Schein SoftLine Chairside Reline Material is a denture reline acrylic which is self-curing and fast setting and provides a soft, non-irritating tissue bearing surface. It decreases patient discomfort and faults such as crazing and warping. Henry Schein Softline Reline Material produces a very dense and non-porous reline which is accomplished by the extra fine granules of powder and cross-linking liquid. It provides a strong reline material which is highly resistant to the absorption of stains and odors.

INDICATIONS FOR USE

Henry Schein Softline is used for temporary relining of the tissue bearing surface of the acrylic of full and partial removable dentures when the tissue needs a soft surface to facilitate healing or when the patient cannot tolerate a hard acrylic surface on the mucosa.

FORMULATION

The liquid and primer are primarily mono-methacrylate monomers; the powder is long chain polymethacrylate beads; the combination contains a cold curing catalyst system.

DIRECTIONS FOR USE

- Clean the denture with soap and water. Trim high spots and irritating rough surfaces and remove a thin layer from the tissue bearing surface of the denture using an acrylic bur. Do not reduce the border height but abrade slightly with the bur.
- To assure a good seal between the liner and the denture base, apply a coat of Acrylic Bonding Primer.
- When bonding is not desired on surfaces of acrylic teeth, interproximal areas, outer ridges, acrylic filling materials, jackets, bridges or opposing dentures, apply water soluble silicone lubricant for protection.
- Measure the quantity of powder and liquid using the two red vials provided. The small vial is for the liquid. The large vial is for the Pink Powder. Fill the vials to the top with each material. Pour the liquid into the mixing cup, then add the powder.
- Using a stainless steel spatula, mix thoroughly for 30 seconds. Allow the mixture to "gel" to the desired consistency, approximately 2 minutes. Spread the material from the crest of the ridge to the periphery covering the postdam area and borders. For a lower denture, reline all tissue contact areas.
- Insert the denture and have the patient close his/her mouth lightly in centric position. Press the denture back to reduce excessive labial bulk. Have the patient bite gently to a comfortable position and proper vertical dimension. To obtain a functional fit, have the patient suck in the cheeks and purse the lips.
- Remove the denture at 3 minutes after insertion. If there are any voids, more freshly mixed material can be added at this time.
- Replace the denture in the mouth for an additional 3 minutes, keeping the teeth in relaxed occlusion.
- Bench cure for 20 minutes before washing with cold water.
- Softline has a rubber-like consistency after setting.
- A slow turning lathe and special burs for rubbery materials may be used for firming and polishing. Continuous cooling of the surface may be required during this finishing process.

WARNINGS

DO NOT allow the denture to remain in the mouth more than 3 minutes during curing because the exothermic reaction could cause burning and irritation. The Acrylic Bonding Primer and Liquid contain methacrylates which are highly flammable and irritating to eyes, respiratory system and skin. Handle with protective gloves

If contact with eyes occur, flush with copious amounts of water and seek medical advice.

If either liquid contacts the skin, wash with soap and water.

In case of ingestion, seek medical attention immediately.

Any non-sterilizable item that is handled in the delivery of the dental service, should be disinfected by standard dental office hygiene procedures.

The liquids are highly volatile and flammable. Keep away from any source of ignition.

Do not use in patients with allergy to these materials.

Do not empty into drains.

STORAGE

Replace caps and lids immediately after use.

Store liquids in well ventilated areas.

The system is designed to be stored between 50°F - 75°F (10°C-24°C).

Shelf life is 2 years.

Do not store in the proximity of eugenol-containing products. Do not store in intense light.

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Henry Schein U.K. Holdings Ltd.