HENRY SCHEIN® DIAMOND POLISHING PASTE

For Professional Use Only

Instructions For Use

Indications:

This product contains actual diamond particles. Designed for polishing porcelain, gold, base metals and composite restorations to give a natural, lifelike appearance. Recommended for lab or intraoral applications.

Porcelain use:

After adjusting porcelain margins with diamond burs, white stones and assorted rubber instruments use this diamond polishing paste to create a very smooth, highly polished surface. NOTE: Clinical testing has proven that highly polished porcelain surfaces are easier to maintain and resist plaque accumulation.

Cast Metal Use:

After initial polishing with stones, discs or rubber points, this diamond polishing paste can be lightly buffed for final luster.

Composite use:

After initial polishing with discs or points, use this diamond polishing paste for final finishing of composite restorations. Dispense small amount on mixing pad. Apply prophyl cup onto paste and polish.

Warnings:

- Dentists and assistants should wear protective eye wear. Patients also should wear eye protection.
- If contact with eyes occurs, flush with copious amounts of water and seek medical advice.
- Any non-sterilizable item that is handled in the delivery of the dental service should be disinfected by standard dental office hygiene procedures.

Storage:

Store at normal room temperature 65°-74°F (18°-24°C). It is important that caps are tightly replaced after each use.

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