# HENRY SCHEIN® ONE STEP

## HCG URINE TEST • PREGNANCY TEST

### Test HCG • de embarazo

### HCG test urinaire • de grossesse

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9008561 Strip Test/tira de prueba/bandelettes de test/Teststreifen

9008562 Cassette Test/casete de prueba/test de la cassette/Testkassette


9008563 Midstream/Medio de la corriente de prueba/mi-chemin de test/Midstream-Test

#### CB One Step HCG Urine Test

One Step HCG Urine Test is a rapid pregnancy test, which you can easily carry out yourself. It detects the presence of human chorionic gonadotropin ("hCG"), which appears in urine very early during pregnancy. For external use only. For professional use only.

**HOW DOES IT WORK?**  
Human chorionic gonadotropin (hCG) is a hormone, produced by the developing placenta shortly after the conception and secreted into the urine. The pregnancy test contains antibodies which specifically react with this hormone.  
When the strip is immersed into a urine specimen, capillary action carries the specimen to migrate along the membrane. When hCG in the sample reaches the test zone region of the membrane, it will form a coloured line. Absence of this coloured line suggests a negative result.  
To serve as a procedure control, a coloured line will appear at the control zone region, if the test has been correctly performed.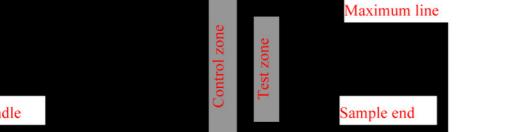

#### CONTENT OF THE TEST KIT

1. 1 x pouch containing a reaction strip and a desiccant. The desiccant is for storage purposes only, and is not used in the test procedures.
2. Instructions for use.

#### WHAT ELSE IS REQUIRED?

1. A clean, dry, plastic or glass container to collect the urine.
2. Timer (watch or clock)

#### SPECIMEN COLLECTION AND PREPARATION

##### WHEN TO COLLECT URINE FOR THE TEST?

Any urine specimen is appropriate for pregnancy testing but the first morning urine specimen is optimal because of its highest concentration of HCG.


##### HOW TO COLLECT URINE?

Urine specimens may be collected in any clean, dry, plastic or glass container.

#### 9008561 – HS One Step Pregnancy urine strip test

##### HOW TO DO THE TEST?

1. The test strip and urine have to be at room temperature (15-30°C) for testing.
2. Remove the test strip from the sealed pouch.
3. Do not urinate past the arrow mark and not passed the arrow mark. Take the strip out after 3 seconds and lay the strip flat on a clean, dry, non-absorbent surface (such as the edge of the urine container).
4. Positive results may be observed at around 1 minute. However, to confirm negative results, the complete reaction time of 3 minutes is required. Do not read results after 5 minutes as they may be inaccurate.

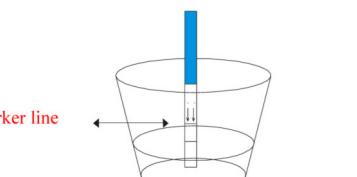#### HOW TO READ THE RESULTS

##### Negative (not pregnant):

Only one colour band appears, in the control zone. No apparent band on the test zone. This indicates that no pregnancy has been detected.

##### Positive (pregnant):

Distinct colour bands appear in the control and test zones. It indicates that you are pregnant. The colour intensity of the test bands may vary since different stages of pregnancy have different concentrations of hCG hormone.

##### Invalid:

No visible band at all, or there is a visible band only in the test region but not in the control region. Repeat with a new test kit. If test still fails, please contact the distributor, where you bought the product, with the lot number.

NOTE: if the colour in the test zone is weak, it is recommended that the test be repeated in 48 hours.

#### 9008562 – HS One Step Pregnancy urine cassette test

##### HOW TO DO THE TEST?

1. Remove a testing device from the foil pouch by tearing at the notch and place it on a level surface.
2. Holding a sample dropper vertically, add exactly four drops of the urine specimen to the sample well (with an arrow marked).
3. Positive results may be observed at around 1 minute. However, to confirm negative results, the complete reaction time of 3 minutes is required. Do not read results after 5 minutes as the results may be inaccurate.

#### HOW TO READ THE RESULTS?

##### Negative (not pregnant):

Only one colour band appears, in the control zone. No apparent band on the test zone. This indicates that no pregnancy has been detected.

##### Positive (pregnant):

Distinct colour bands appear in the control and test zones. It indicates that you are pregnant. The colour intensity of the test bands may vary since different stages of pregnancy have different concentrations of hCG hormone.

##### Invalid:

No visible band at all, or there is a visible band only in the test region but not in the control region. Repeat with a new test kit. If test still fails, please contact the distributor, where you bought the product, with the lot number.

To serve as a procedure control, a coloured line will appear at the control zone region, if the test has been correctly performed.


NOTE: if the colour in the test zone is weak, it is recommended that the test be repeated in 48 hours.

#### 9008563 – HS One Step Pregnancy urine midstream test

##### HOW TO DO THE TEST?

1. To begin testing, open the sealed pouch by tearing along the notch. Remove test device--midstream from the pouch.
2. Hold the round end of cover with one hand. Use the other hand to pull out the midstream cap and expose the absorbent.
3. Point the absorbent tip downward; place the absorbent tip in urine stream for at least 10 seconds to be thoroughly wet. Do not urinate past the arrow mark. Otherwise, you can collect your urine into a clean cup and dip half of the absorbent pad into the urine for at least 3 seconds.
4. Re-cap the device and wait for colour bands to appear. Positive results may be observed at around 1 minute. However, to confirm negative results, the complete reaction time of 3 minutes is required. Do not read results after 5 minutes as the results may be inaccurate.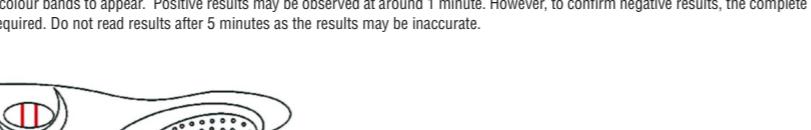

#### HOW TO READ THE RESULTS?

##### Negative (not pregnant):


Only one colour band appears, in the control zone. No apparent band on the test zone. This indicates that no pregnancy has been detected.

##### Positive (pregnant):

Distinct colour bands appear in the control and test zones. It indicates that you are pregnant. The colour intensity of the test bands may vary since different stages of pregnancy have different concentrations of hCG hormone.

##### Invalid:

No visible band at all, or there is a visible band only in the test region but not in the control region. Repeat with a new test kit. If test still fails, please contact the distributor, where you bought the product, with the lot number.

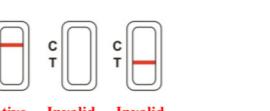NOTE: if the colour in the test zone is weak, it is recommended that the test be repeated in 48 hours.

#### WHAT TO DO WHEN YOU KNOW THE RESULT

##### IF THE RESULT IS POSITIVE (TWO LINES)

Consult your physician. He should confirm your pregnancy after evaluating all clinical and laboratory findings. Indeed, in certain conditions, the test may give a false positive result. He will also give you further information and advice on your pregnancy.

See also "Limitations".

##### IF THE RESULT IS NEGATIVE (1 LINE)

If you have reasons to believe that you are pregnant (e.g. no menstrual bleeding), you can repeat the test a few days later. If the test is still negative, consult your physician. See also "Limitations".

##### Invalid:

No visible band at all, or there is a visible band only in the test region but not in the control region. Repeat with a new test kit. If test still fails, please contact the distributor, where you bought the product, with the lot number.

See also "Limitations".

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1. As it is with any diagnostic procedure, a confirmed pregnancy diagnosis should only be made by a physician after evaluating all clinical and laboratory findings.
2. If a urine sample is too dilute (i.e., low specific gravity) it may not contain a representative level of hCG. If pregnancy is still suspected, another urine specimen should be collected 48 hours later and tested.
3. Low concentration of hCG in a very early pregnancy can give a negative result. In this case, another specimen should be obtained at least 48 hours later and tested.
4. Elevated levels of hCG can be caused by a few conditions other than pregnancy. Therefore, the presence of hCG in urine specimen should not be used to diagnose pregnancy unless these conditions have been ruled out.
5. A normal pregnancy cannot be distinguished from an ectopic pregnancy based on hCG levels alone. Also, spontaneous miscarriage may cause confusion in interpreting test results.
6. Sterility treatments, based on hCG, may cause false results. Consult your physician.

#### EXPECTED RESULTS

Negative results are expected in healthy non-pregnant women and healthy men. Although the hCG levels in normal early pregnant women are variable, One Step HCG Urine Test is capable of detecting pregnancy as early as 1 day after the first missed menses.

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### HOW TO DO THE TEST?

1. To begin testing, open the sealed pouch by tearing along the notch. Remove test device--midstream from the pouch.
2. Hold the round end of cover with one hand. Use the other hand to pull out the midstream cap and expose the absorbent.
3. Point the absorbent tip downward; place the absorbent tip in urine stream for at least 10 seconds to thoroughly wet. Do not urinate past the arrow mark. Otherwise, you can collect your urine into a clean cup and dip half of the absorbent pad into the urine for at least 3 seconds.
4. Re-cap the device and wait for color bands to appear. Positive results may be observed in as soon as 1 minute. However, to confirm negative results, the complete reaction time of 3 minutes is required. Do not read results after 5 minutes.

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